



Working With You to Deliver Smiles... Industry Best Solutions Since 1933

SPECIALISTS IN APPLIANCE THERAPY FOR ADULTS AND CHILDREN

ADJUSTMENT INSTRUCTIONS FOR: THE BIONATOR

INSERTION OF APPLIANCE:

1. The appliance should fit loose when inserted. It does not “snap” into place
2. Very little adjustment will be needed.
3. The labial wire should be 1/2mm away from the anterior teeth.
4. See the patient 3-4 days after inserting the appliance to check to see how the patient is progressing and if there are any problems wearing the appliance.
5. See the patient one week later to monitor how they are wearing the appliance and make sure they are not have any problems.

ADJUSTMENTS TO BE MADE AT EACH APPOINTMENT:

1. Each month you will need to see the patient to check their progress.
2. The labial wire should remain about 1/2mm away from the anterior teeth.
3. The Coffin Spring should be off the tissue.
4. If needed the expansion screw should be turned one turn once a week.
5. Be sure to cut the lingual wire if you are going to have the patient turn the screw.
6. Begin removing interproximal acrylic on the lower posterior teeth to increase the vertical. Acrylic can be removed from the distal of the upper posteriors. Be sure to leave acrylic mesial of the upper 1st molars especially in the mixed dentition.

PATIENT INSTRUCTIONS:

1. This appliance should be worn 20-22 hours a day. It should be removed during eating.
2. To clean the appliance, brush it after each meal with tooth paste.

FOR ADDITIONAL ADJUSTMENT INFORMATION ABOUT THIS APPLIANCE
AND MANY MORE, VISIT OUR YOUTUBE CHANNEL AT:

www.youtube.com/OhlendorfApplianceLab

2840 CLARK AVENUE • P.O. BOX 7212 • ST. LOUIS, MO 63177 • 1-800-325-8921

e-mail address - info@OhlendorfApplianceLab.com Web address - www.OhlendorfApplianceLab.com